

ergotron®

Healthcare Cleaning Recommendations

You can use most hospital-grade, diluted, non-abrasive, non-corrosive cleaning solutions on all plastic and metal surfaces, whether painted, polished or unfinished. See the following pages for a list of tested cleaning solutions by product family.

These are cleaning recommendations only and do not replace your cleaning and infection control requirements. Please consult your infection control administrator or epidemiologist for all cleaning and infection control procedures and processes.

Helpful tips:

- Be sure to test any cleaning solution on a small, hidden area before widespread use.
- Pen and permanent and dry erase markers should be removed with 91% Isopropyl alcohol (generic) and a soft cloth.
- Iodine stains can be removed with commonly used cleaners and a soft cloth.

For your safety and to protect your equipment:

- To avoid the risk of electric shock, do not expose electrical components to water, cleaning solutions or other potentially corrosive liquids or substances.
- Do not use flammable cleaners on cart surfaces due to the proximity to electrical power.
- Never use steel wool or other abrasive materials that will damage the surface finish.
- Do not use strong solvents such as trichloroethylene and acetone as they can damage the surface finish.
- Do not immerse the cart or cart components in liquid or allow liquids to flow into the cart. Wipe all cleaners off the surface immediately using a damp cloth and dry the surface after cleaning.



To support infection control, please clean, sanitize and sterilize all Ergotron equipment prior to first use and frequently while in use, in accordance with applicable health organization and government guidelines. This includes, but is not limited to, the World Health Organization, the Centers for Disease Control and Infection, the National Institutes of Health and healthcare industry guidelines. Guidelines are evolving, so we recommend checking these sites frequently.

Your local Customer Care team is available to help answer any questions about this information:
USA & Canada: 800.888.8458 / +1.651.681.7600 / customerservice@ergotron.com
EMEA: +31.33.45.45.600 / info.eu@ergotron.com
APAC: apaccustomerservice@ergotron.com
LATAM: +1.800.681.1820 / info.latam@ergotron.com
Custom: custom@ergotron.com

© Ergotron, Inc. 07.23.2025
Literature made in the U.S.A.
Content subject to change.
Ergotron devices are not intended to cure, treat, mitigate or prevent any disease.
Patent information available at www.ergotron.com/patents.

TABLE OF CONTENTS

| | |
|---------------------------------------|----|
| Mosaic | 3 |
| CareFit™ | 5 |
| Carts | 6 |
| Combos | 7 |
| Enclosures | 9 |
| StyleView® | 11 |
| Laptop, LCD Pivot, Pole Carts | 12 |
| LCD Arm Carts | 13 |
| SV10 Carts | 15 |
| LCD Pivot Cart with Large Worksurface | 17 |
| Combos, White | 19 |
| Combos, Polished Aluminum | 21 |
| Sit-Stand Vertical Lifts | 23 |
| Appendix | 25 |



How to Use This Guide

This guide was developed to help customers understand the compatibility of select cleaning agents that have been specifically tested on Ergotron healthcare solutions. Please note, our internal testing focused on the cosmetic impact of the listed solutions and did not account for potential structural changes over time or their effectiveness in cleaning or disinfecting the products.

Active ingredients between similar cleaning agents may vary. Check the relevant manufacturer's website for the current active ingredients list, and be sure to test a small area of the product before applying a cleaning agent to the full solution. Cleaning solutions with similar ingredients listed in this guide may also be compatible. Conduct your own testing before use.

As always, follow the cleaning agent manufacturer's directions, and consult your infection control administrator or epidemiologist for all cleaning procedures and processes. We will update our guide as further testing is complete using additional cleaning agents and products.

KEY

Part numbers:

"X" placeholders represent various drawer configurations and global power options.

Example:

C52-12XX-X applies to C52-1201-1, as well as C52-1211-3.

Product part colors:

Match the designated color for each product part with the product image to understand the tested area.

Compatible:

Internal Ergotron testing showed no cosmetic changes to product parts with selected cleaning agent.

May cause cosmetic changes to material:

Minimal cosmetic changes may become visible after prolonged use; other solutions are recommended.

Not compatible:

Internal Ergotron testing showed cosmetic changes after use; not recommended.

Not tested:

The cleaning agent has not yet been tested on the specified Ergotron product part.

Contact your Ergotron representative or local Customer Care team with any questions about this information. Please refer to page 1 for contact details.



PRODUCT FAMILY: MOSAIC
PRODUCT TYPES: CARTS (LAPTOP, LCD - WHITE/WARM GREY)

| Category | Cleaning Agent | | Product Part (if applicable) | | | | | | | |
|-------------------------------|--|--------|------------------------------|-------------------------|-----------------------|---------|------------------|---------|--------|------|
| | Brand | Type | Arm / Tilt / Riser | Worksurface / Head Unit | Keyboard & Mouse Tray | Handles | Levers / Buttons | Drawers | Column | Base |
| Alcohol | Isopropyl alcohol (generic), 99% | Liquid | ✓ | ✓ | ✓ | ✓ | ✓ | ✓ | ✓ | ✓ |
| | Sterillium® Comfort Gel™, Hand Antiseptic | Liquid | ✓ | ✓ | ✓ | ✓ | * | ✓ | * | * |
| Ammonia | Windex®, SC Johnson Professional® | Liquid | ✓ | ✓ | ✓ | ✓ | ✓ | ✓ | ✓ | ✓ |
| IPA, Quarternary Ammonium | Super Sani-Cloth®, PDI | Wipes | ✓ | ✓ | ✓ | ✓ | * | ✓ | ✓ | * |
| | Sani-Cloth® Plus | Wipes | ✓ | ✓ | ✓ | ✓ | ✓ | ✓ | ✓ | ✓ |
| Quarternary Ammonium | CaviWipes™, Metrex | Wipes | ✓ | ✓ | ✓ | ✓ | ✓ | ✓ | ✓ | ✓ |
| | Clinell® BCW200 Universal Wipes (green) | Wipes | ✓ | ✓ | ✓ | ✓ | ✓ | ✓ | ✓ | ✓ |
| | Clorox® Versasure® Wipes | Wipes | ✓ | ✓ | ✓ | ✓ | ✓ | ✓ | ✓ | ✓ |
| Ethanol, Quarternary Ammonium | Lysol® Disinfectant Spray | Spray | ✓ | * | ✓ | ✓ | * | ✓ | ✓ | * |
| Bleach | Clorox Healthcare® Bleach Germicidal Wipes | Wipes | * | * | * | ✓ | ✓ | * | ✓ | * |
| Hydrogen Peroxide | Diversey™ Oxivir® Tb Wipes, Virox Technologies | Wipes | ✓ | * | * | ✓ | ✓ | * | ✓ | * |
| | Clorox Healthcare® Hydrogen Peroxide Wipes | Wipes | ✓ | ✓ | ✓ | ✓ | ✓ | ✓ | ✓ | ✓ |
| | Hydrogen Peroxide | Liquid | ✓ | ✓ | ✓ | ✓ | ✓ | ✓ | ✓ | ✓ |
| Peracetic | Clinell® Peracetic Acid Wipes CS25 (red) | Wipes | ✓ | ✓ | ✓ | ✓ | ✓ | ✓ | ✓ | ✓ |

Product Compatibility

Touch Display



If cleaning is necessary, Isopropyl alcohol or Ethyl alcohol, either 100% or mixed 50%/50% with distilled water, may be used on a moist clean soft cloth. Do not use ketones and aromatic solvents which will damage the polarizer materials.

Compatible ✓

May cause cosmetic changes to material *

Not compatible

Not tested

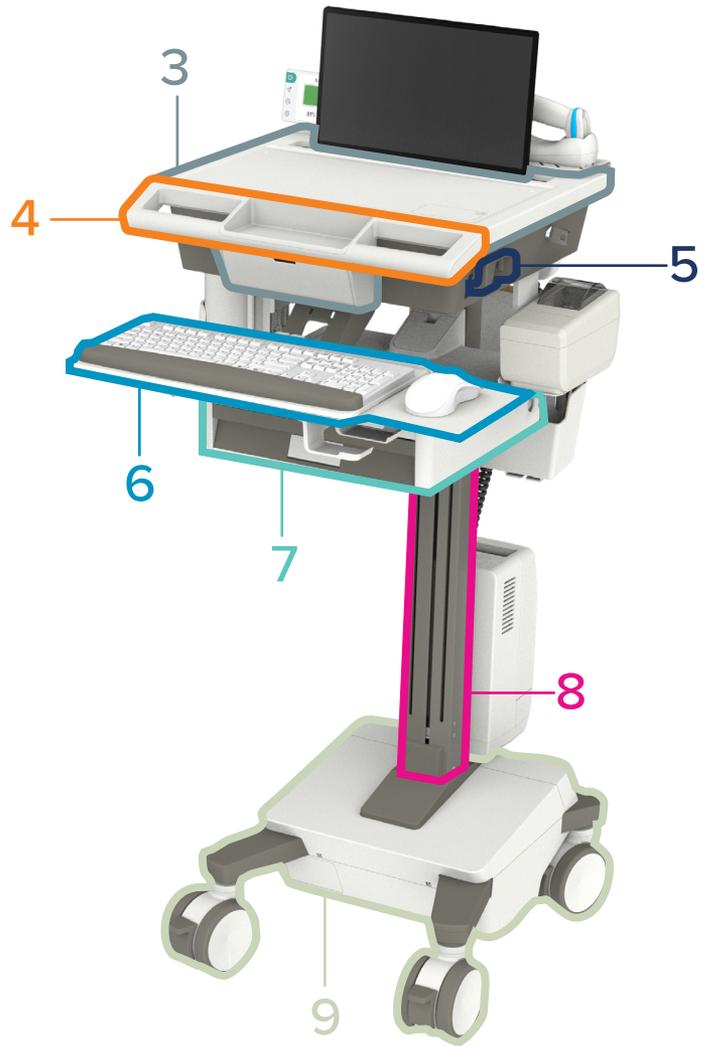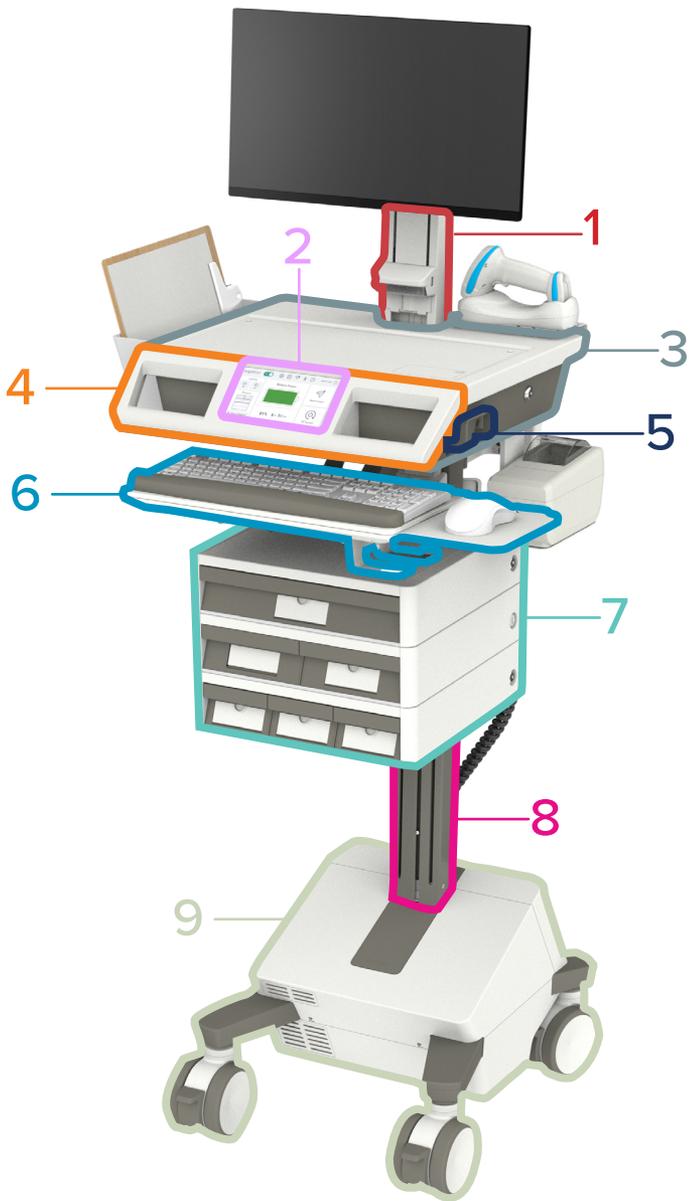

- | | | | | | | | | |
|---|---|-------------------------|---|---|-----------------------|---|---|---------|
| 1 |  | Arm / Tilt / Riser | 4 |  | Handles | 7 |  | Drawers |
| 2 |  | Touch Display | 5 |  | Levers / Buttons | 8 |  | Column |
| 3 |  | Worksurface / Head Unit | 6 |  | Keyboard & Mouse Tray | 9 |  | Base |

PRODUCT FAMILY: CAREFIT




PRODUCT TYPES: CARTS (WHITE/WARM GREY)

- C50-1100-0
- C50-3500-0
- C50-3510-0
- C52-12XX-X
- C52-22XX-X

| Category | Cleaning Agent | | Product Part (if applicable) | | | | | | | |
|-------------------------------|--|--------|------------------------------|-------------------------|-----------------------|---------|------------------|---------|--------|------|
| | Brand | Type | Arm / Tilt / Riser | Worksurface / Head Unit | Keyboard & Mouse Tray | Handles | Levers / Buttons | Drawers | Column | Base |
| Alcohol | Isopropyl alcohol (generic), 99% | Liquid | ✓ | ✓ | ✓ | ✓ | ✓ | ✓ | ✓ | ✓ |
| | Sterillium® Comfort Gel™, Hand Antiseptic | Liquid | ✓ | ✓ | ✓ | ✓ | ✓ | ✓ | ✓ | ✓ |
| Ammonia | Windex®, SC Johnson Professional® | Liquid | ✓ | ✓ | ✓ | ✓ | ✓ | ✓ | ✓ | ✓ |
| IPA, Quarternary Ammonium | Super Sani-Cloth®, PDI | Wipes | ✓ | ✓ | ✓ | ✓ | ✓ | ✓ | ✓ | ✓ |
| | Sani-Cloth® Plus | Wipes | ✓ | ✓ | ✓ | ✓ | ✓ | ✓ | ✓ | ✓ |
| Quarternary Ammonium | CaviWipes™, Metrex | Wipes | ✓ | ✓ | ✓ | ✓ | ✓ | ✓ | ✓ | ✓ |
| | Clinell® BCW200 Universal Wipes (green) | Wipes | ✓ | ✓ | ✓ | ✓ | ✓ | ✓ | ✓ | ✓ |
| | Clorox® Versasure® Wipes | Wipes | * | ✓ | ✓ | ✓ | * | ✓ | ✓ | * |
| Ethanol, Quarternary Ammonium | Lysol® Disinfectant Spray | Spray | ✓ | ✓ | ✓ | ✓ | ✓ | ✓ | ✓ | ✓ |
| Bleach | Clorox Healthcare® Bleach Germicidal Wipes | Wipes | ✓ | ✓ | ✓ | ✓ | ✓ | ✓ | ✓ | ✓ |
| Hydrogen Peroxide | Diversey™ Oxivir® Tb Wipes, Virox Technologies | Wipes | ✓ | * | * | ✓ | ✓ | * | ✓ | ✓ |
| | Clorox Healthcare® Hydrogen Peroxide Wipes | Wipes | ✓ | ✓ | ✓ | ✓ | ✓ | ✓ | ✓ | ✓ |
| | Hydrogen Peroxide | Liquid | ✓ | ✓ | ✓ | ✓ | ✓ | ✓ | ✓ | ✓ |
| Peracetic | Clinell® Peracetic Acid Wipes CS25 (red) | Wipes | * | ✓ | ✓ | ✓ | * | ✓ | ✓ | * |

Product Compatibility

| | |
|--|---|
| Compatible | ✓ |
| May cause cosmetic changes to material | * |
| Not compatible | ■ |
| Not tested | ■ |

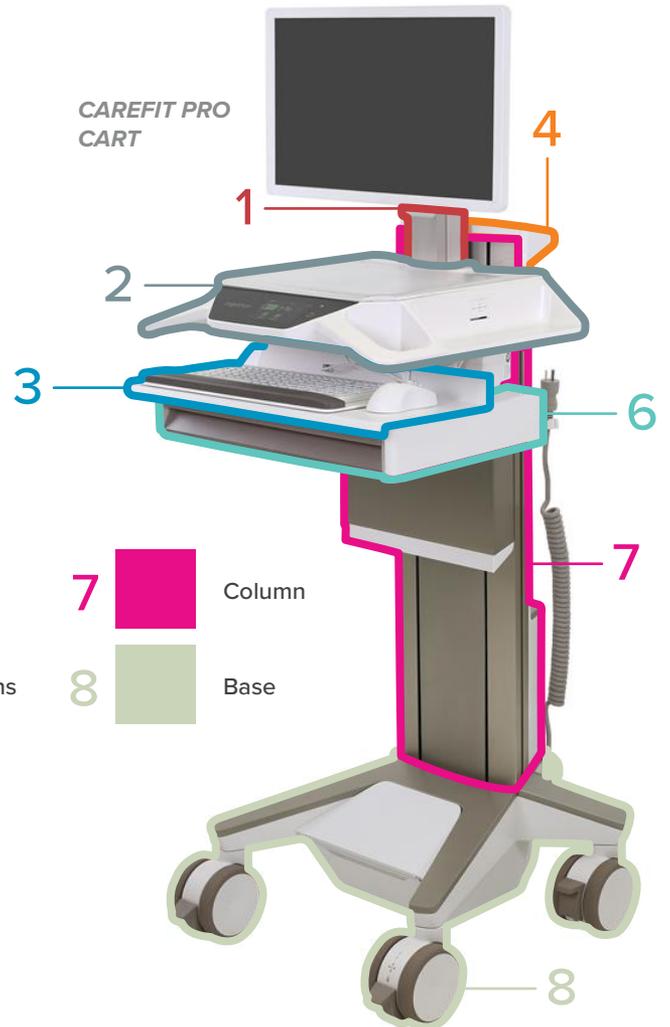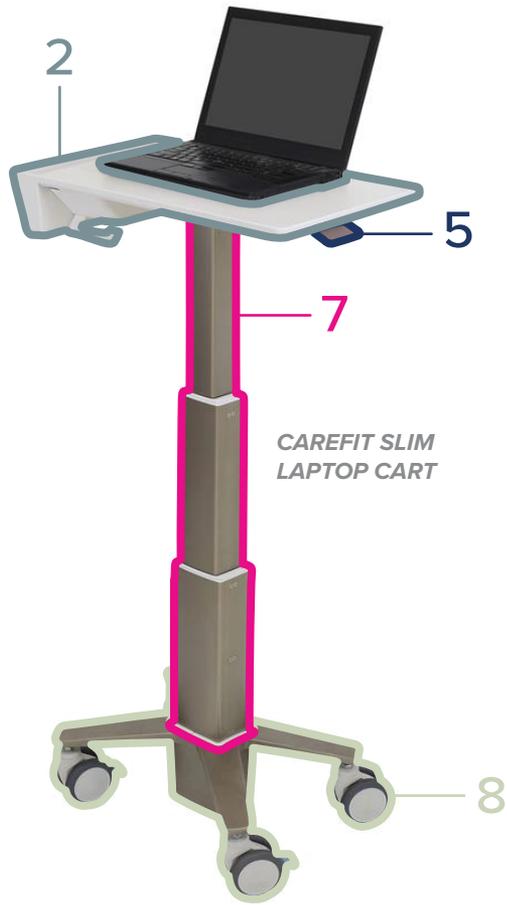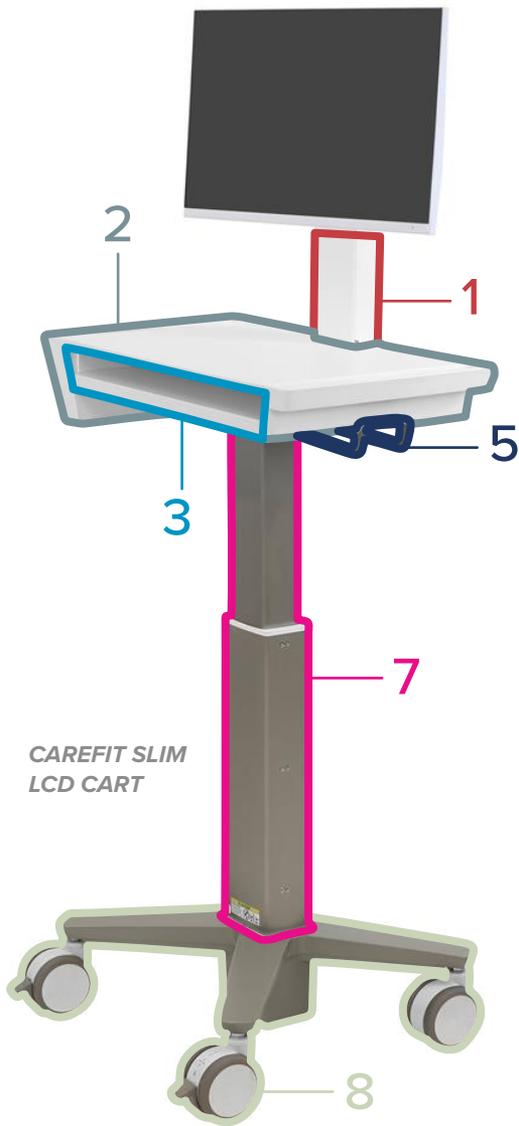

- | | | | | | | | | |
|---|---|-------------------------|---|---|------------------|---|--|--------|
| 1 |  | Arm / Tilt / Riser | 4 |  | Handles | 7 |  | Column |
| 2 |  | Worksurface / Head Unit | 5 |  | Levers / Buttons | 8 |  | Base |
| 3 |  | Keyboard & Mouse Tray | 6 |  | Drawers | | | |

PRODUCT FAMILY: CAREFIT
PRODUCT TYPES: COMBOS (WHITE)

45-618-251
 45-619-251
 45-621-251
 45-622-251

| Category | Cleaning Agent | | Product Part (if applicable) | | | | | |
|-------------------------------|--|--------|------------------------------|-----|-------------|-----------------------|--|-------------------|
| | Brand | Type | Tilt / Riser | Arm | Worksurface | Keyboard & Mouse Tray | Wall Track <small>Wall Track Covers hide the Wall Track</small> | Wall Track Covers |
| Alcohol | Isopropyl alcohol (generic), 99% | Liquid | ✓ | ✓ | ✓ | ✓ | ✓ | ✓ |
| | Sterillium® Comfort Gel™, Hand Antiseptic | Liquid | ✓ | ✓ | ✓ | ✓ | ✓ | ✓ |
| Ammonia | Windex®, SC Johnson Professional® | Liquid | ✓ | ✓ | ✓ | ✓ | ✓ | ✓ |
| IPA, Quarternary Ammonium | Super Sani-Cloth®, PDI | Wipes | ✓ | ✓ | ✓ | ✓ | ✓ | ✓ |
| | Sani-Cloth® Plus | Wipes | ✓ | ✓ | ✓ | ✓ | ✓ | ✓ |
| Quarternary Ammonium | CaviWipes™, Metrex | Wipes | ✓ | ✓ | ✓ | ✓ | ✓ | ✓ |
| | Clinell® BCW200 Universal Wipes (green) | Wipes | ✓ | ✓ | ✓ | ✓ | ✓ | ✓ |
| | Clorox® Versasure® Wipes | Wipes | ✓ | ✓ | ✓ | ✓ | ✓ | ✓ |
| Ethanol, Quarternary Ammonium | Lysol® Disinfectant Spray | Spray | ✓ | ✓ | ✓ | ✓ | ✓ | ✓ |
| Bleach | Clorox Healthcare® Bleach Germicidal Wipes | Wipes | ✓ | ✓ | ✓ | ✓ | ✓ | ✓ |
| Hydrogen Peroxide | Diversey™ Oxivir® Tb Wipes, Virox Technologies | Wipes | * | * | * | * | * | * |
| | Clorox Healthcare® Hydrogen Peroxide Wipes | Wipes | ✓ | ✓ | ✓ | ✓ | ✓ | ✓ |
| | Hydrogen Peroxide | Liquid | ✓ | ✓ | ✓ | ✓ | ✓ | ✓ |
| Peracetic | Clinell® Peracetic Acid Wipes CS25 (red) | Wipes | ✓ | ✓ | ✓ | ✓ | ✓ | ✓ |

Product Compatibility


Compatible ✓

May cause cosmetic changes to material *

Not compatible

Not tested

PRODUCT FAMILY: CAREFIT
PRODUCT TYPES: COMBOS (WHITE)

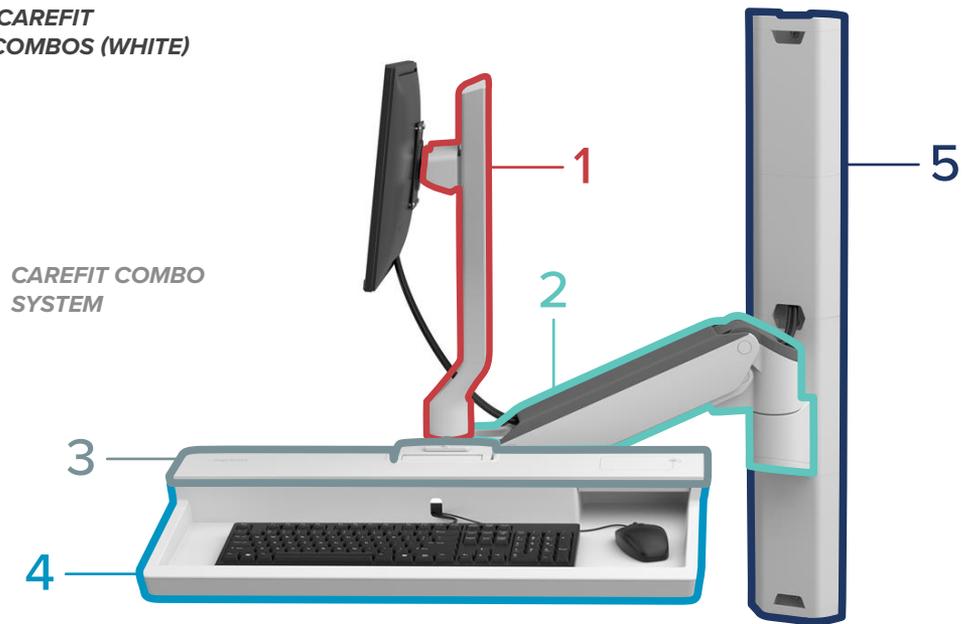

- | | | | | | |
|---|--|--------------|---|---|-----------------------|
| 1 |  | Tilt / Riser | 4 |  | Keyboard & Mouse Tray |
| 2 |  | Arm | 5 |  | Wall Track Covers |
| 3 |  | Worksurface | | | |

PRODUCT FAMILY: CAREFIT
PRODUCT TYPES: ENCLOSURES (BLACK OR WHITE)

61-367-060
 61-367-030

| Category | Cleaning Agent | | Product Part (if applicable) | | | | | |
|-------------------------------|--|--------|------------------------------|---------|-------------|-----------------------|--------|-------------|
| | Brand | Type | Lift Engine | Handles | Worksurface | Keyboard & Mouse Tray | Covers | Wall Covers |
| Alcohol | Isopropyl alcohol (generic), 99% | Liquid | ✓ | ✓ | ✓ | ✓ | ✓ | ✓ |
| | Sterillium® Comfort Gel™, Hand Antiseptic | Liquid | ✓ | ✓ | ✓ | ✓ | ✓ | ✓ |
| Ammonia | Windex®, SC Johnson Professional® | Liquid | ✓ | ✓ | ✓ | ✓ | ✓ | ✓ |
| IPA, Quarternary Ammonium | Super Sani-Cloth®, PDI | Wipes | ✓ | ✓ | ✓ | ✓ | ✓ | ✓ |
| | Sani-Cloth® Plus | Wipes | ✓ | ✓ | ✓ | ✓ | ✓ | ✓ |
| Quarternary Ammonium | CaviWipes™, Metrex | Wipes | ✓ | ✓ | ✓ | ✓ | ✓ | ✓ |
| | Clinell® BCW200 Universal Wipes (green) | Wipes | ✓ | ✓ | ✓ | ✓ | ✓ | ✓ |
| | Clorox® Versasure® Wipes | Wipes | ✓ | ✓ | ✓ | ✓ | ✓ | ✓ |
| Ethanol, Quarternary Ammonium | Lysol® Disinfectant Spray | Spray | ✓ | ✓ | ✓ | ✓ | ✓ | ✓ |
| Bleach | Clorox Healthcare® Bleach Germicidal Wipes | Wipes | ✓ | ✓ | ✓ | ✓ | ✓ | ✓ |
| Hydrogen Peroxide | Diversey™ Oxivir® Tb Wipes, Virox Technologies | Wipes | ✓ | * | * | * | ✓ | ✗ |
| | Clorox Healthcare® Hydrogen Peroxide Wipes | Wipes | ✓ | ✓ | ✓ | ✓ | ✓ | ✓ |
| | Hydrogen Peroxide | Liquid | ✓ | ✓ | ✓ | ✓ | ✓ | ✓ |
| Peracetic | Clinell® Peracetic Acid Wipes CS25 (red) | Wipes | ✓ | ✓ | ✓ | ✓ | ✓ | ✓ |

Product Compatibility

Compatible 

May cause cosmetic changes to material 

Not compatible 

Not tested 

CAREFIT ENCLOSURE
WALL-MOUNT COMPUTER
WORKSTATION



- | | | | | | |
|---|---|-------------|---|---|-----------------------|
| 1 |  | Lift Engine | 4 |  | Keyboard & Mouse Tray |
| 2 |  | Handles | 5 |  | Covers |
| 3 |  | Worksurface | 6 |  | Wall Covers |

PRODUCT FAMILY: STYLEVIEW

PRODUCT TYPES: CARTS (LAPTOP, LCD PIVOT, POLE – WHITE)

SV40-6100-0 SV42-8302-X
 SV40-6300-0 SV43-11XX-X
 SV41-6100-0 SV43-13XX-X
 SV41-6300-0 SV44-11XX-X
 SV42-6102-X SV44-13XX-X
 SV42-6302-X 24-818-211

| Category | Cleaning Agent | | Product Part (if applicable) | | | | | | | |
|-------------------------------|--|--------|------------------------------|-------------------------|-----------------------|---------|------------------|---------|--------|------|
| | Brand | Type | Arm / Tilt / Riser | Worksurface / Head Unit | Keyboard & Mouse Tray | Handles | Levers / Buttons | Drawers | Column | Base |
| Alcohol | Isopropyl alcohol (generic), 99% | Liquid | ✓ | ✓ | ✓ | ✓ | * | ✓ | ✓ | * |
| | Sterillium® Comfort Gel™, Hand Antiseptic | Liquid | ✓ | ✓ | ✓ | ✓ | ✓ | ✓ | ✓ | ✓ |
| Ammonia | Windex®, SC Johnson Professional® | Liquid | ✓ | ✓ | ✓ | ✓ | ✓ | ✓ | ✓ | ✓ |
| IPA, Quarternary Ammonium | Super Sani-Cloth®, PDI | Wipes | ✓ | ✓ | ✓ | ✓ | * | ✓ | ✓ | * |
| | Sani-Cloth® Plus | Wipes | ✓ | ✓ | ✓ | ✓ | * | ✓ | ✓ | * |
| Quarternary Ammonium | CaviWipes™, Metrex | Wipes | ✓ | ✓ | ✓ | ✓ | * | ✓ | ✓ | * |
| | Clinell® BCW200 Universal Wipes (green) | Wipes | ✓ | ✓ | ✓ | ✓ | * | ✓ | ✓ | * |
| | Clorox® Versasure® Wipes | Wipes | ✓ | ✓ | ✓ | ✓ | * | ✓ | ✓ | * |
| Ethanol, Quarternary Ammonium | Lysol® Disinfectant Spray | Spray | ✓ | ✓ | ✓ | ✓ | ✓ | ✓ | ✓ | ✓ |
| Bleach | Clorox Healthcare® Bleach Germicidal Wipes | Wipes | ✓ | ✓ | ✓ | ✓ | * | ✓ | ✓ | * |
| Hydrogen Peroxide | Diversey™ Oxivir® Tb Wipes, Virox Technologies | Wipes | ✓ | ✓ | ✓ | ✓ | * | ✓ | ✓ | * |
| | Clorox Healthcare® Hydrogen Peroxide Wipes | Wipes | ✓ | ✓ | ✓ | ✓ | * | ✓ | ✓ | * |
| | Hydrogen Peroxide | Liquid | ✓ | ✓ | ✓ | ✓ | ✓ | ✓ | ✓ | ✓ |
| Peracetic | Clinell® Peracetic Acid Wipes CS25 (red) | Wipes | ✓ | ✓ | ✓ | ✓ | * | ✓ | ✓ | * |

Product Compatibility



Compatible 

May cause cosmetic changes to material 

Not compatible 


Not tested 

STYLEVIEW LCD CART

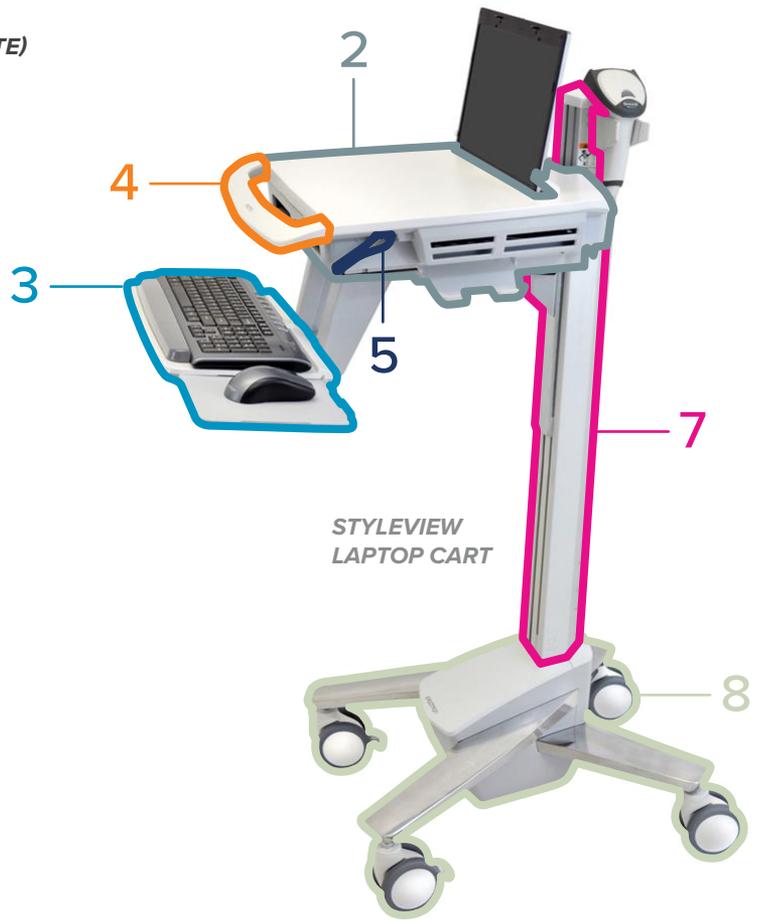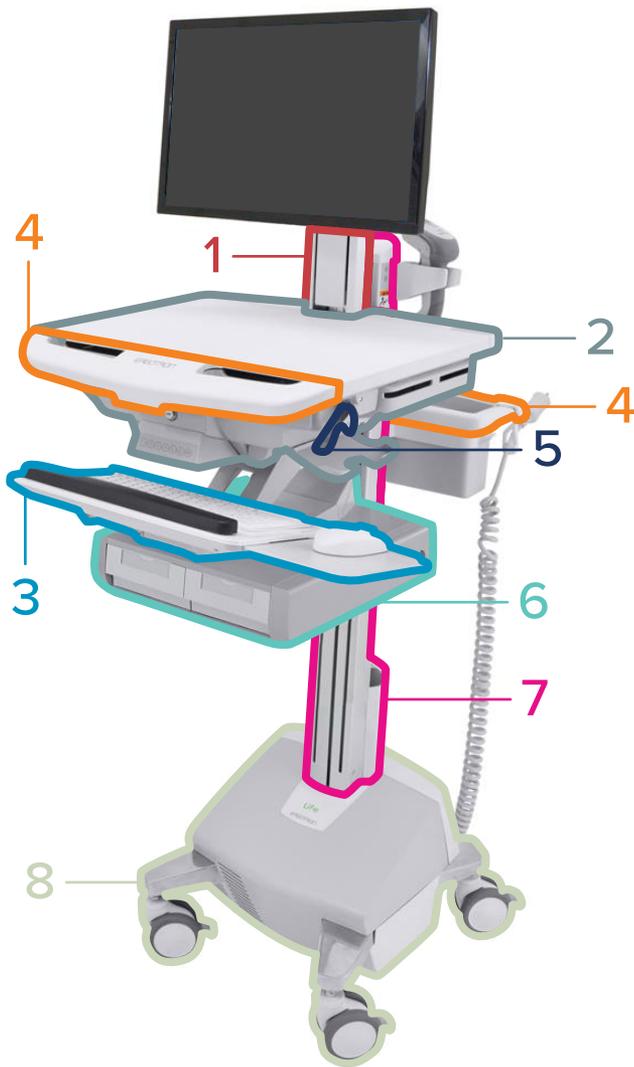

STYLEVIEW
LAPTOP CART

STYLEVIEW
LCD PIVOT CART

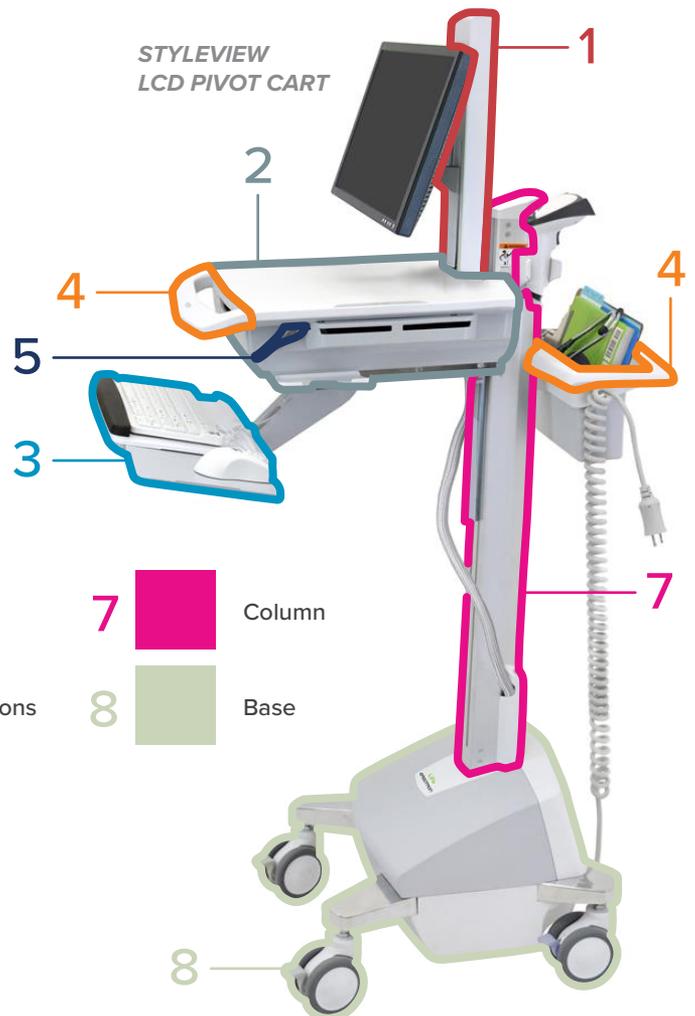

- | | | | | | | | | |
|----------|---|-------------------------|----------|---|------------------|----------|--|--------|
| 1 |  | Arm / Tilt / Riser | 4 |  | Handles | 7 |  | Column |
| 2 |  | Worksurface / Head Unit | 5 |  | Levers / Buttons | 8 |  | Base |
| 3 |  | Keyboard & Mouse Tray | 6 |  | Drawers | | | |

PRODUCT FAMILY: STYLEVIEW

PRODUCT TYPES: CARTS (LCD ARM — WHITE)


SV41-6200-0
 SV42-6202-X
 SV43-12XX-X
 SV44-12XX-X

| Category | Cleaning Agent | | Product Part (if applicable) | | | | | | | |
|-------------------------------|--|--------|------------------------------|-------------------------|-----------------------|---------|------------------|---------|--------|------|
| | Brand | Type | Arm / Tilt / Riser | Worksurface / Head Unit | Keyboard & Mouse Tray | Handles | Levers / Buttons | Drawers | Column | Base |
| Alcohol | Isopropyl alcohol (generic), 99% | Liquid | * | ✓ | ✓ | ✓ | * | ✓ | ✓ | * |
| | Sterillium® Comfort Gel™, Hand Antiseptic | Liquid | ✓ | ✓ | ✓ | ✓ | ✓ | ✓ | ✓ | ✓ |
| Ammonia | Windex®, SC Johnson Professional® | Liquid | ✓ | ✓ | ✓ | ✓ | ✓ | ✓ | ✓ | ✓ |
| IPA, Quarternary Ammonium | Super Sani-Cloth®, PDI | Wipes | * | ✓ | ✓ | ✓ | * | ✓ | ✓ | * |
| | Sani-Cloth® Plus | Wipes | * | ✓ | ✓ | ✓ | * | ✓ | ✓ | * |
| Quarternary Ammonium | CaviWipes™, Metrex | Wipes | * | ✓ | ✓ | ✓ | * | ✓ | ✓ | * |
| | Clinell® BCW200 Universal Wipes (green) | Wipes | * | ✓ | ✓ | ✓ | * | ✓ | ✓ | * |
| | Clorox® Versasure® Wipes | Wipes | * | ✓ | ✓ | ✓ | * | ✓ | ✓ | * |
| Ethanol, Quarternary Ammonium | Lysol® Disinfectant Spray | Spray | ✓ | ✓ | ✓ | ✓ | ✓ | ✓ | ✓ | ✓ |
| Bleach | Clorox Healthcare® Bleach Germicidal Wipes | Wipes | * | ✓ | ✓ | ✓ | * | ✓ | ✓ | * |
| Hydrogen Peroxide | Diversey™ Oxivir® Tb Wipes, Virox Technologies | Wipes | ✓ | ✓ | ✓ | ✓ | * | | ✓ | * |
| | Clorox Healthcare® Hydrogen Peroxide Wipes | Wipes | * | ✓ | ✓ | ✓ | * | ✓ | ✓ | * |
| | Hydrogen Peroxide | Liquid | ✓ | ✓ | ✓ | ✓ | ✓ | ✓ | ✓ | ✓ |
| Peracetic | Clinell® Peracetic Acid Wipes CS25 (red) | Wipes | * | ✓ | ✓ | ✓ | * | ✓ | ✓ | * |


Product Compatibility

| | |
|--|---|
| Compatible | ✓ |
| May cause cosmetic changes to material | * |
| Not compatible | ■ |
| Not tested | ■ |

PRODUCT FAMILY: STYLEVIEW
 PRODUCT TYPES: CARTS (LCD ARM — WHITE)

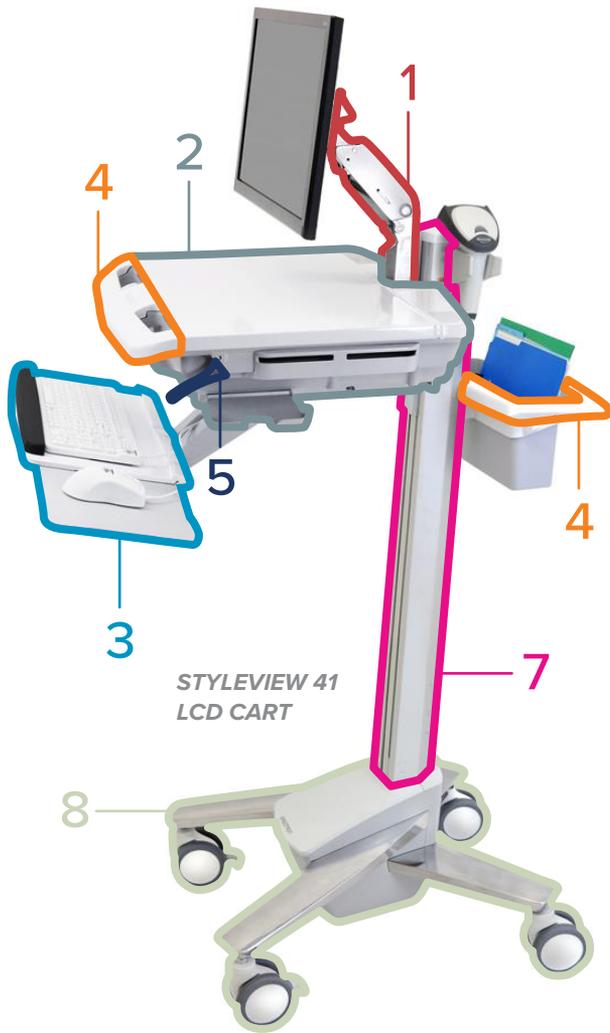

STYLEVIEW 44
 LCD CART

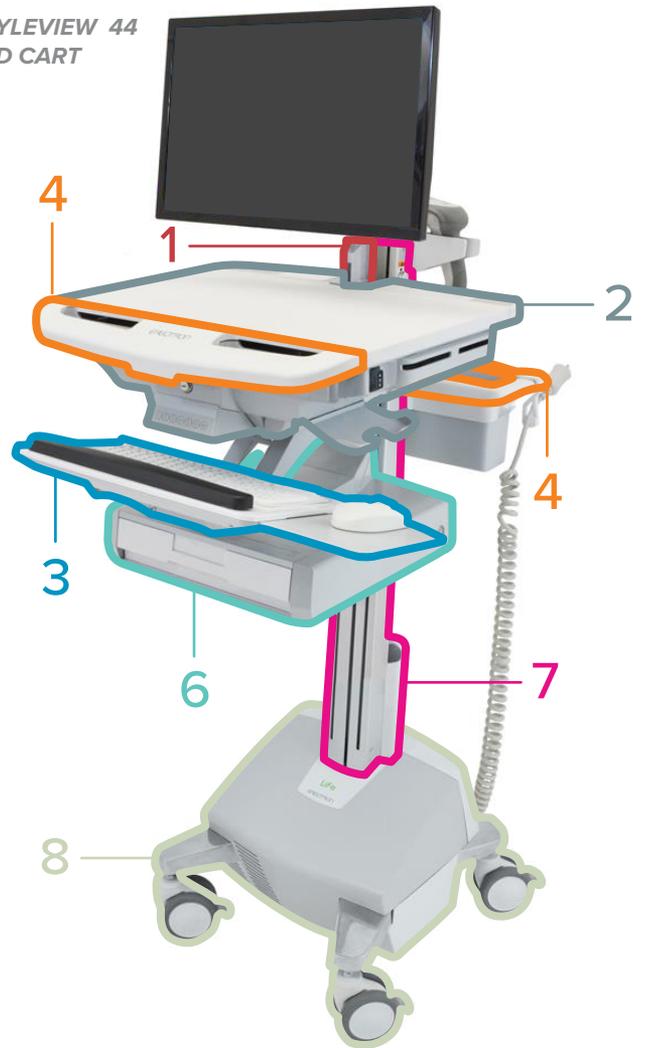

- | | | | | | | | | |
|---|---|-------------------------|---|---|------------------|---|---|--------|
| 1 |  | Arm / Tilt / Riser | 4 |  | Handles | 7 |  | Column |
| 2 |  | Worksurface / Head Unit | 5 |  | Levers / Buttons | 8 |  | Base |
| 3 |  | Keyboard & Mouse Tray | 6 |  | Drawers | | | |

PRODUCT FAMILY: STYLEVIEW
PRODUCT TYPES: CARTS (WHITE)

SV10-1100-0
 SV10-1400-0
 SV10-1800-0

| Category | Cleaning Agent | | Product Part (if applicable) | | | | | |
|-------------------------------|--|--------|------------------------------|-------------------------|---------|------------------|--------|------|
| | Brand | Type | Arm / Tilt / Riser | Worksurface / Head Unit | Handles | Levers / Buttons | Column | Base |
| Alcohol | Isopropyl alcohol (generic), 99% | Liquid | ✓ | ✓ | ✓ | ✓ | ✓ | ✓ |
| | Sterillium® Comfort Gel™, Hand Antiseptic | Liquid | ✓ | ✓ | ✓ | ✓ | ✓ | ✓ |
| Ammonia | Windex®, SC Johnson Professional® | Liquid | ✓ | ✓ | ✓ | ✓ | ✓ | ✓ |
| IPA, Quarternary Ammonium | Super Sani-Cloth®, PDI | Wipes | ✓ | ✓ | ✓ | ✓ | ✓ | ✓ |
| | Sani-Cloth® Plus | Wipes | ✓ | ■ | ✓ | ✓ | ✓ | ✓ |
| Quarternary Ammonium | CaviWipes™, Metrex | Wipes | ✓ | ✓ | ✓ | ✓ | ✓ | ✓ |
| | Clinell® BCW200 Universal Wipes (green) | Wipes | ✓ | ■ | ✓ | ✓ | ✓ | ✓ |
| | Clorox® Versasure® Wipes | Wipes | ✓ | ■ | ✓ | ✓ | ✓ | ✓ |
| Ethanol, Quarternary Ammonium | Lysol® Disinfectant Spray | Spray | ✓ | ■ | ✓ | ✓ | ✓ | ✓ |
| Bleach | Clorox Healthcare® Bleach Germicidal Wipes | Wipes | ✓ | ✓ | ✓ | ✓ | ✓ | ✓ |
| Hydrogen Peroxide | Diversey™ Oxivir® Tb Wipes, Virox Technologies | Wipes | ✓ | ✓ | ✓ | * | ✓ | * |
| | Clorox Healthcare® Hydrogen Peroxide Wipes | Wipes | ✓ | ✓ | ✓ | ✓ | ✓ | ✓ |
| | Hydrogen Peroxide | Liquid | ✓ | ■ | ✓ | ✓ | ✓ | ✓ |
| Peracetic | Clinell® Peracetic Acid Wipes CS25 (red) | Wipes | ✓ | ■ | ✓ | ✓ | ✓ | ✓ |


Product Compatibility

Compatible 

May cause cosmetic changes to material 

Not compatible 

Not tested 

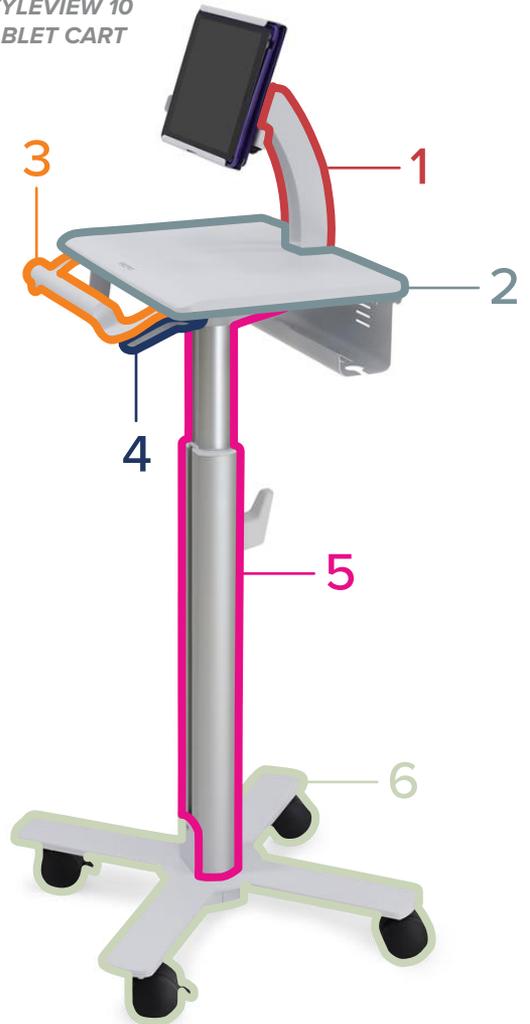

- | | | | | | |
|---|---|-------------------------|---|--|------------------|
| 1 |  | Arm / Tilt / Riser | 4 |  | Levers / Buttons |
| 2 |  | Worksurface / Head Unit | 5 |  | Column |
| 3 |  | Handles | 6 |  | Base |

PRODUCT FAMILY: STYLEVIEW
PRODUCT TYPES: LCD PIVOT CART WITH LARGE WORKSURFACE
(WHITE)

SV41-6320

| Category | Cleaning Agent | | Product Part (if applicable) | | | | | |
|-------------------------------|--|--------|------------------------------|-------------------------|---------|------------------|--------|------|
| | Brand | Type | Arm / Tilt / Riser | Worksurface / Head Unit | Handles | Levers / Buttons | Column | Base |
| Alcohol | Isopropyl alcohol (generic), 99% | Liquid | ✓ | | ✓ | * | ✓ | * |
| | Sterillium® Comfort Gel™, Hand Antiseptic | Liquid | ✓ | | ✓ | ✓ | ✓ | ✓ |
| Ammonia | Windex®, SC Johnson Professional® | Liquid | ✓ | | ✓ | ✓ | ✓ | ✓ |
| IPA, Quarternary Ammonium | Super Sani-Cloth®, PDI | Wipes | ✓ | | ✓ | * | ✓ | * |
| | Sani-Cloth® Plus | Wipes | ✓ | | ✓ | * | ✓ | * |
| Quarternary Ammonium | CaviWipes™, Metrex | Wipes | ✓ | | ✓ | * | ✓ | * |
| | Clinell® BCW200 Universal Wipes (green) | Wipes | ✓ | | ✓ | * | ✓ | * |
| | Clorox® Versasure® Wipes | Wipes | ✓ | | ✓ | * | ✓ | * |
| Ethanol, Quarternary Ammonium | Lysol® Disinfectant Spray | Spray | ✓ | | ✓ | ✓ | ✓ | ✓ |
| Bleach | Clorox Healthcare® Bleach Germicidal Wipes | Wipes | ✓ | | ✓ | * | ✓ | * |
| Hydrogen Peroxide | Diversey™ Oxivir® Tb Wipes, Virox Technologies | Wipes | ✓ | ✓ | ✓ | * | ✓ | * |
| | Clorox Healthcare® Hydrogen Peroxide Wipes | Wipes | ✓ | | ✓ | * | ✓ | * |
| | Hydrogen Peroxide | Liquid | ✓ | | ✓ | ✓ | ✓ | ✓ |
| Peracetic | Clinell® Peracetic Acid Wipes CS25 (red) | Wipes | ✓ | | ✓ | * | ✓ | * |

Product Compatibility


Compatible ✓

May cause cosmetic changes to material *

Not compatible

Not tested

PRODUCT FAMILY: STYLEVIEW
PRODUCT TYPES: LCD PIVOT CART WITH LARGE WORKSURFACE
(WHITE)

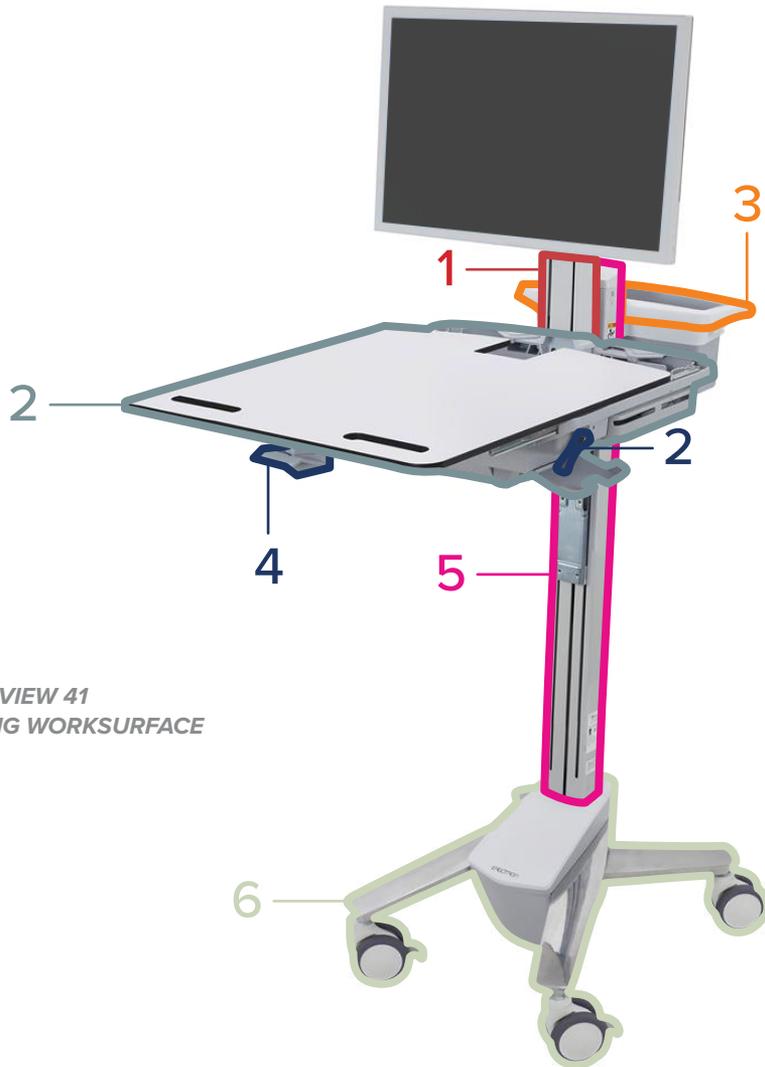

STYLEVIEW 41
SLIDING WORKSURFACE
CART

- | | | | | | |
|---|---|-------------------------|---|--|------------------|
| 1 |  | Arm / Tilt / Riser | 4 |  | Levers / Buttons |
| 2 |  | Worksurface / Head Unit | 5 |  | Column |
| 3 |  | Handles | 6 |  | Base |

PRODUCT FAMILY: STYLEVIEW

PRODUCT TYPES: COMBOS (WHITE)

- 45-583-216 45-272-216
- 45-260-216 45-270-216
- 45-266-216 45-273-216
- 45-594-216 45-271-216
- 45-595-216

| Category | Cleaning Agent | | Product Part (if applicable) | | | | | |
|-------------------------------|--|--------|------------------------------|-----|-------------|-----------------------|------------|-------------------|
| | Brand | Type | Tilt / Riser | Arm | Worksurface | Keyboard & Mouse Tray | Wall Track | Wall Track Covers |
| Alcohol | Isopropyl alcohol (generic), 99% | Liquid | ✓ | ✓ | ✓ | ✓ | ✓ | ✓ |
| | Sterillium® Comfort Gel™, Hand Antiseptic | Liquid | ✓ | ✓ | ✓ | ✓ | ✓ | ✓ |
| Ammonia | Windex®, SC Johnson Professional® | Liquid | ✓ | ✓ | ✓ | ✓ | ✓ | ✓ |
| IPA, Quarternary Ammonium | Super Sani-Cloth®, PDI | Wipes | ✓ | ✓ | ✓ | ✓ | ✓ | ✓ |
| | Sani-Cloth® Plus | Wipes | ✓ | ✓ | ✓ | ✓ | ✓ | ✓ |
| Quarternary Ammonium | CaviWipes™, Metrex | Wipes | ✓ | ✓ | ✓ | ✓ | ✓ | ✓ |
| | Clinell® BCW200 Universal Wipes (green) | Wipes | ✓ | ✓ | ✓ | ✓ | ✓ | ✓ |
| | Clorox® Versasure® Wipes | Wipes | ✓ | ✓ | ✓ | ✓ | ✓ | ✓ |
| Ethanol, Quarternary Ammonium | Lysol® Disinfectant Spray | Spray | ✓ | ✓ | ✓ | ✓ | ✓ | ✓ |
| Bleach | Clorox Healthcare® Bleach Germicidal Wipes | Wipes | ✓ | ✓ | ✓ | ✓ | ✓ | ✓ |
| Hydrogen Peroxide | Diversey™ Oxivir® Tb Wipes, Virox Technologies | Wipes | * | ✓ | * | * | ✓ | ✓ |
| | Clorox Healthcare® Hydrogen Peroxide Wipes | Wipes | ✓ | ✓ | ✓ | ✓ | ✓ | ✓ |
| | Hydrogen Peroxide | Liquid | ✓ | ✓ | ✓ | ✓ | ✓ | ✓ |
| Peracetic | Clinell® Peracetic Acid Wipes CS25 (red) | Wipes | ✓ | ✓ | ✓ | ✓ | ✓ | ✓ |


Product Compatibility

Compatible 

May cause cosmetic changes to material 

Not compatible 

Not tested 

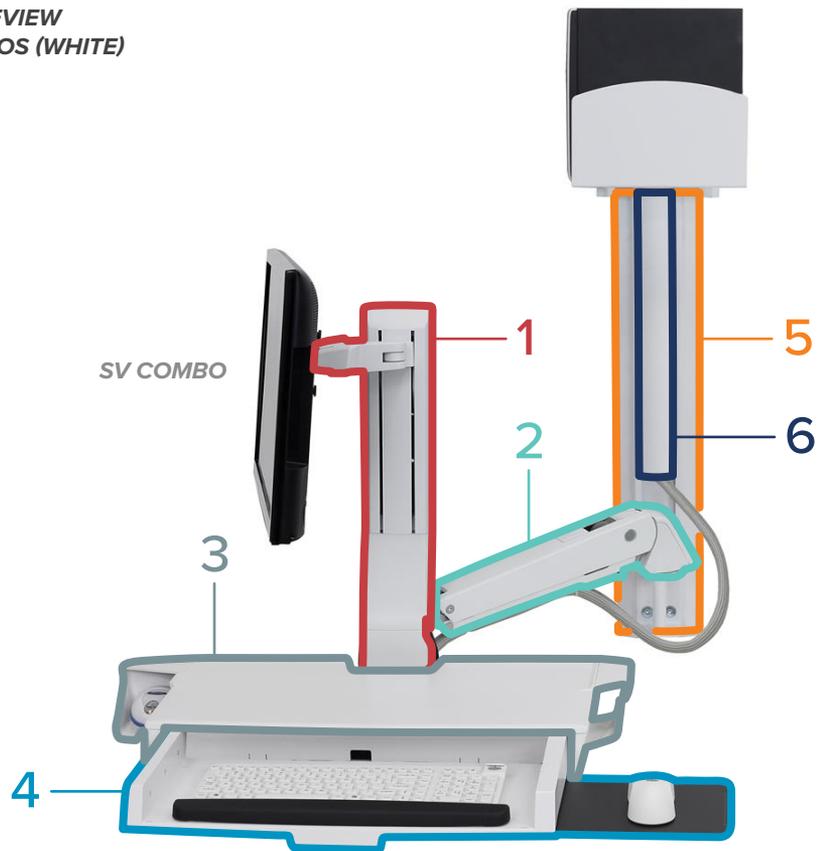

- | | | | | | |
|---|---|--------------|---|---|-----------------------|
| 1 |  | Tilt / Riser | 4 |  | Keyboard & Mouse Tray |
| 2 |  | Arm | 5 |  | Wall Track |
| 3 |  | Worksurface | 6 |  | Wall Track Covers |

PRODUCT FAMILY: STYLEVIEW

PRODUCT TYPES: COMBOS (POLISHED ALUMINUM)

- 45-583-026 45-272-026
- 45-260-026 45-270-026
- 45-266-026 45-273-026
- 45-594-026 45-271-026
- 45-595-026

| Category | Cleaning Agent | | Product Part (if applicable) | | | | | |
|-------------------------------|--|--------|------------------------------|-----|-------------|-----------------------|------------|-------------------|
| | Brand | Type | Tilt / Riser | Arm | Worksurface | Keyboard & Mouse Tray | Wall Track | Wall Track Covers |
| Alcohol | Isopropyl alcohol (generic), 99% | Liquid | ✓ | * | ✓ | ✓ | * | ✓ |
| | Sterillium® Comfort Gel™, Hand Antiseptic | Liquid | ✓ | ✓ | ✓ | ✓ | ✓ | ✓ |
| Ammonia | Windex®, SC Johnson Professional® | Liquid | ✓ | ✓ | ✓ | ✓ | ✓ | ✓ |
| IPA, Quarternary Ammonium | Super Sani-Cloth®, PDI | Wipes | ✓ | * | ✓ | ✓ | * | ✓ |
| | Sani-Cloth® Plus | Wipes | ✓ | * | ✓ | ✓ | * | ✓ |
| Quarternary Ammonium | CaviWipes™, Metrex | Wipes | ✓ | * | ✓ | ✓ | * | ✓ |
| | Clinell® BCW200 Universal Wipes (green) | Wipes | ✓ | * | ✓ | ✓ | * | ✓ |
| | Clorox® Versasure® Wipes | Wipes | ✓ | * | ✓ | ✓ | * | ✓ |
| Ethanol, Quarternary Ammonium | Lysol® Disinfectant Spray | Spray | ✓ | ✓ | ✓ | ✓ | ✓ | ✓ |
| Bleach | Clorox Healthcare® Bleach Germicidal Wipes | Wipes | ✓ | * | ✓ | ✓ | * | ✓ |
| Hydrogen Peroxide | Diversey™ Oxivir® Tb Wipes, Virox Technologies | Wipes | ✗ | ✗ | ✗ | ✗ | ✗ | ✓ |
| | Clorox Healthcare® Hydrogen Peroxide Wipes | Wipes | ✓ | * | ✓ | ✓ | * | ✓ |
| | Hydrogen Peroxide | Liquid | ✓ | ✓ | ✓ | ✓ | ✓ | ✓ |
| Peracetic | Clinell® Peracetic Acid Wipes CS25 (red) | Wipes | ✓ | * | ✓ | ✓ | * | ✓ |

Product Compatibility


Compatible 

May cause cosmetic changes to material 

Not compatible 

Not tested 

PRODUCT FAMILY: STYLEVIEW
PRODUCT TYPES: COMBOS (POLISHED ALUMINUM)

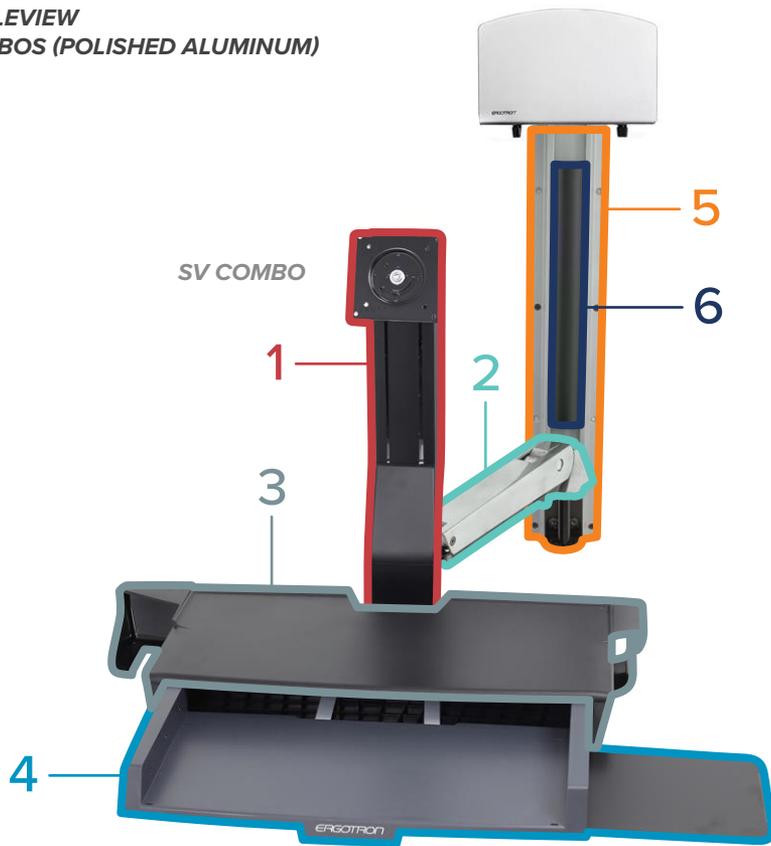

- | | | | | | |
|---|---|--------------|---|---|-----------------------|
| 1 |  | Tilt / Riser | 4 |  | Keyboard & Mouse Tray |
| 2 |  | Arm | 5 |  | Wall Track |
| 3 |  | Worksurface | 6 |  | Wall Track Covers |

PRODUCT FAMILY: STYLEVIEW

PRODUCT TYPES: SIT-STAND VERTICAL LIFTS (WHITE OR BLACK)

61-080-062

61-080-085

61-081-062

61-081-085

| Category | Cleaning Agent | | Product Part (if applicable) | | | |
|-------------------------------|--|--------|------------------------------|---------|-------------|-----------------------|
| | Brand | Type | Lift Engine | Handles | Worksurface | Keyboard & Mouse Tray |
| Alcohol | Isopropyl alcohol (generic), 99% | Liquid | ✓ | ✓ | ✓ | ✓ |
| | Sterillium® Comfort Gel™, Hand Antiseptic | Liquid | ✓ | ✓ | ✓ | ✓ |
| Ammonia | Windex®, SC Johnson Professional® | Liquid | ✓ | ✓ | ✓ | ✓ |
| IPA, Quarternary Ammonium | Super Sani-Cloth®, PDI | Wipes | ✓ | ✓ | ✓ | ✓ |
| | Sani-Cloth® Plus | Wipes | ✓ | ✓ | ✓ | ✓ |
| Quarternary Ammonium | CaviWipes™, Metrex | Wipes | ✓ | ✓ | ✓ | ✓ |
| | Clinell® BCW200 Universal Wipes (green) | Wipes | ✓ | ✓ | ✓ | ✓ |
| | Clorox® Versasure® Wipes | Wipes | ✓ | ✓ | ✓ | ✓ |
| Ethanol, Quarternary Ammonium | Lysol® Disinfectant Spray | Spray | ✓ | ✓ | ✓ | ✓ |
| Bleach | Clorox Healthcare® Bleach Germicidal Wipes | Wipes | ✓ | ✓ | ✓ | ✓ |
| Hydrogen Peroxide | Diversey™ Oxivir® Tb Wipes, Virox Technologies | Wipes | * | * | ✓ | * |
| | Clorox Healthcare® Hydrogen Peroxide Wipes | Wipes | ✓ | ✓ | ✓ | ✓ |
| | Hydrogen Peroxide | Liquid | ✓ | ✓ | ✓ | ✓ |
| Peracetic | Clinell® Peracetic Acid Wipes CS25 (red) | Wipes | ✓ | ✓ | ✓ | ✓ |

Product Compatibility

Compatible



May cause cosmetic changes to material



Not compatible





Not tested



PRODUCT FAMILY: STYLEVIEW
PRODUCT TYPES: SIT-STAND VERTICAL LIFTS (WHITE OR BLACK)

STYLEVIEW SIT-STAND VERTICAL LIFT
HIGH-TRAFFIC AREAS

STYLEVIEW SIT-STAND VERTICAL LIFT
PATIENT ROOM

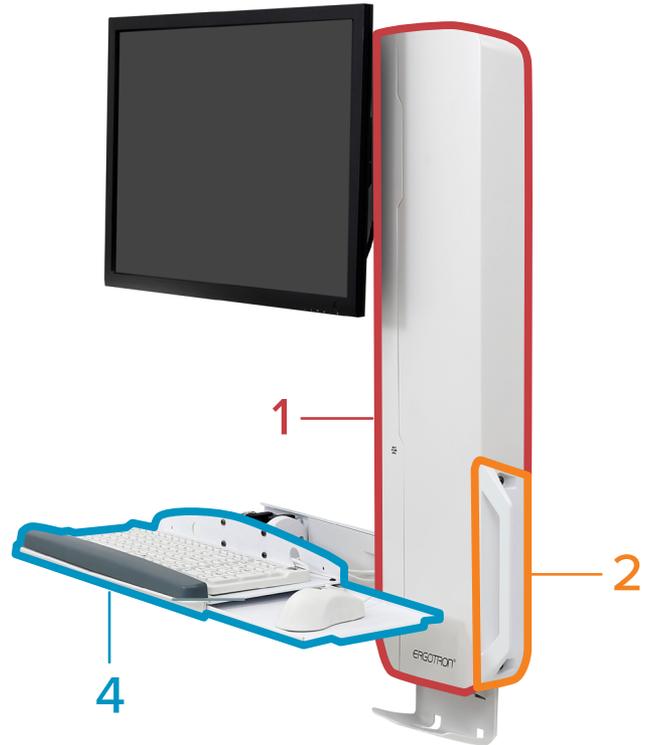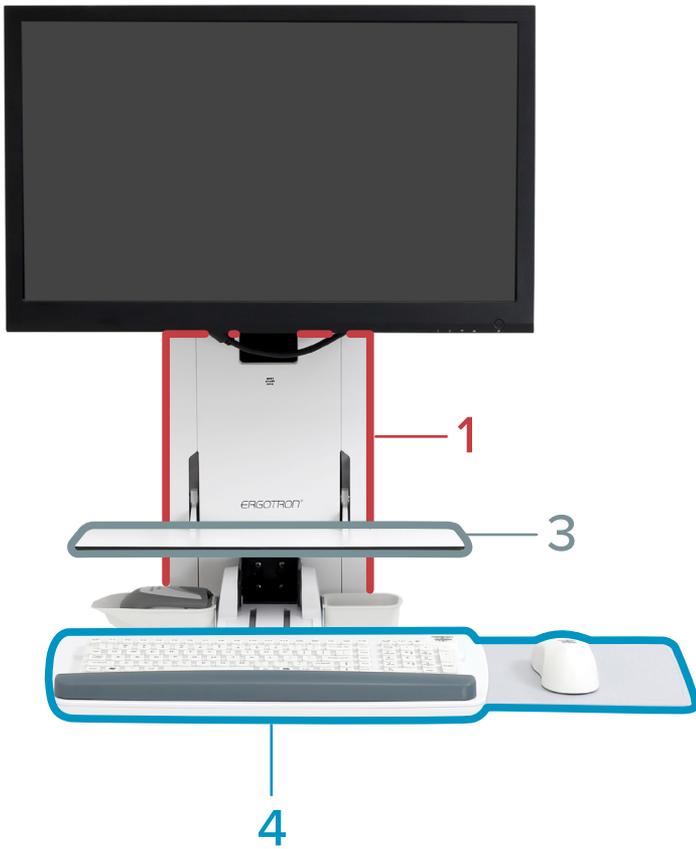

- | | | | | | |
|---|---|-------------|---|---|-----------------------|
| 1 |  | Lift Engine | 3 |  | Worksurface |
| 2 |  | Handles | 4 |  | Keyboard & Mouse Tray |

Appendix

The active ingredients for each cleaning agent at the time of testing are listed below. Active ingredients may change, so be sure to check the relevant manufacturer's website for the current active ingredients list.

| Cleaning Agent | | Active Ingredients |
|--|--------|--|
| Brand | Type | |
| Isopropyl alcohol (generic), 99% | Liquid | Isopropyl alcohol 99% |
| Sterillium® Comfort Gel™, Hand Antiseptic | Liquid | Ethyl alcohol 85% |
| Windex®, SC Johnson Professional® | Liquid | Ammonium Hydroxide, Isopropanolamine, 2-Hexoxyethanol |
| Super Sani-Cloth®, PDI | Wipes | Isopropyl alcohol 55.5%, Quaternary ammonium compounds, C12-18-alkyl [(ethylphenyl) methyl] dimethyl, chlorides 0.25%, Alkyl Dimethyl Benzyl Ammonium Chloride 0.25% |
| Sani-Cloth® Plus | Wipes | Isopropanol 10–20%, 2-Butoxyethanol < 5%, Quaternary ammonium compounds, C12–18-alkyl [(ethylphenyl) methyl] dimethyl, chlorides 0.125%, Benzyl-C12–18-alkyldimethyl ammonium chlorides 0.125% |
| CaviWipes™, Metrex | Wipes | Propan-2-ol, isopropyl alcohol, isopropanol 17.20%, 2-Butoxyethanol, ethylene glycol monobutyl ether, butyl cellosolve 1–5%, Benzethonium Chloride (QA) 0.28% |
| Clinell® BCW200 Universal Wipes (green) | Wipes | 2-Phenoxyethanol 0.6%, Benzalkonium chloride 0.6%, Didecyldimethyl ammonium chloride 0.6%, Biphenyl-2-ol 0.1% |
| Clorox®Versasure® Wipes | Wipes | 0.25% n-Alkyl (C14, 60%; C16, 30%; C12, 5%; C18, 5%) Dimethyl Benzyl Ammonium Chloride, 0.25% n-Alkyl (C12, 68%; C14, 32%) Dimethyl Ethylbenzyl Ammonium Chloride |
| Lysol® Disinfectant Spray | Spray | Ethanol, Alkyl (50% C14, 40% C12, 10% C16) Dimethyl benzyl ammonium saccharinate |
| Clorox Healthcare® Bleach Germicidal Wipes | Wipes | Sodium hypochlorite NaOCl 0.55% |
| Diversey™ Oxivir® Tb Wipes, Virox Technologies | Wipes | Hydrogen peroxide 0.5% |
| Clorox Healthcare® Hydrogen Peroxide Wipes | Wipes | Hydrogen peroxide H2O2 1.4% |
| Hydrogen Peroxide | Liquid | Hydrogen peroxide 3% |
| Clinell® Peracetic Acid Wipes CS25 (red) | Wipes | Sodium Percarbonate ≤50%, Citric Acid ≤15%, Tetraacetylenediamine ≤ 25% |